







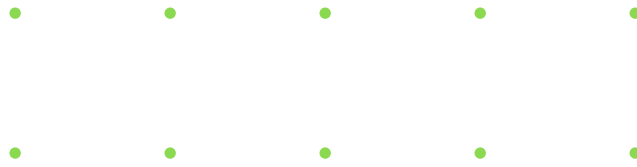


Cavex SiliconA for functional impressions and monophasic impressions

Functional Impression (Putty & Injection Type)

- 1)  Apply Cavex SiliconA Adhesive on the individual tray.
- 2)  Apply strip of Cavex SiliconA Putty. Mixing ratio 1 : 1
- 3)  Massage the cheeks and lips to mould the functional margin.
- 4)  Take impression of the functional margins.
- 5)  Functional margins now defines extent of the muscles and ligaments.
- 6)  Apply Cavex SiliconA Adhesive.
- 7)  Apply Cavex SiliconA Injection Type in the tray and take the impression.
- 8)  Remove the impression from mouth. Disinfect impression (3 min. in Cavex ImpreSafe)

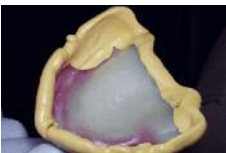



Cavex SiliconA for functional impressions and monophasic impressions
Monophasic Impression (Heavy Body)

- 1)  Implant situation.
- 2)  Fit the individual tray.
- 3)  Apply Cavex SiliconA Adhesive on the individual tray.
- 4)  Apply a strip of Cavex SiliconA Putty. Mixing ratio 1 : 1
- 5)  Massage the cheeks and lips and take the impression of the functional margins.
- 6)  Fill the tray with Cavex SiliconA Heavy Body.
- 7)  Apply Cavex SiliconA Heavy Body around the implants.
- 8)  Place the tray in the mouth.
- 9)  Remove the impression from mouth. Disinfect impression (3 min. in Cavex ImpreSafe)

