








Cavex SiliconA for single stage impressions (Sandwich technique)

Putty & Injection Type

- 1)  Apply Cavex SiliconA Adhesive on a custom tray.
- 2)  Isolate and place retraction cord.
- 3)  Mix Cavex SiliconA Putty. Mixing ratio 1 : 1
- 4)  Knead until the color is homogeneous. Place mixed putty in a tray.
- 5)  Apply Cavex SiliconA Injection Type around the prepared area.
- 6)  Seat the tray firmly, hold motionless and without pressure until the material has set.
- 7)  Remove the impression from mouth. Disinfect impression (3 min. in Cavex ImpreSafe)



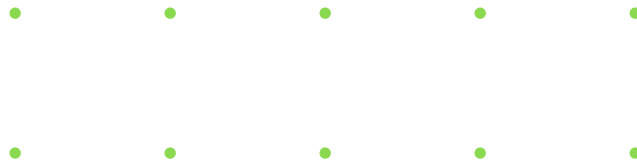

Cavex SiliconA for single stage impressions (Sandwich technique)

Heavy Body & Injection Type

- 1)  Discoloration on central incisor and lateral incisor.
- 2)  Prepare the teeth.
- 3)  Isolate and place retraction cord.
- 4)  Apply Cavex SiliconA Injection Type around the prepared teeth.
- 5)  Apply Cavex SiliconA Adhesive on a tray. Fill the custom tray with Cavex SiliconA Heavy Body. Seat the tray firmly and hold motionless.
- 6)  Remove the impression from mouth. Disinfect impression (3 min. in Cavex ImpreSafe)

