

The recipe for a perfect impression



From powder to paste

Step A

Store the alginate in the light and air-proof Cavex storage container, to keep it in top condition. First stir the alginate powder to loosen.

Recommended dose upper jaw



Recommended dose lower jaw



Step B

Level off the powder in the Cavex measuring scoop for the correct quantity. Add the correct amount of room-temperature water from the Cavex dosing bottle.

Step C

Put the powder and water into the flexible Cavex mixing bowl. Stir for 10 seconds until all the water has been absorbed. Rap the spatula sharply against the flexible side until all the lumps are gone and the paste is nice and smooth.




Cavex alginates and accessories are delivered by your dental depot. For further information on product composition and applications, contact Cavex, on +31 23 530 77 00 - Dental Customer Service, or at www.cavex.nl

Cavex Holland BV
Harmenjansweg 19
P.O. Box 852
2003 RW Haarlem
The Netherlands
T +31 23 530 77 00
F +31 23 535 64 82
dental@cavex.nl
www.cavex.nl

From paste to impression


Step D

Fill the tray with the alginate paste, indicating the occlusion in the full tray with a moistened finger.

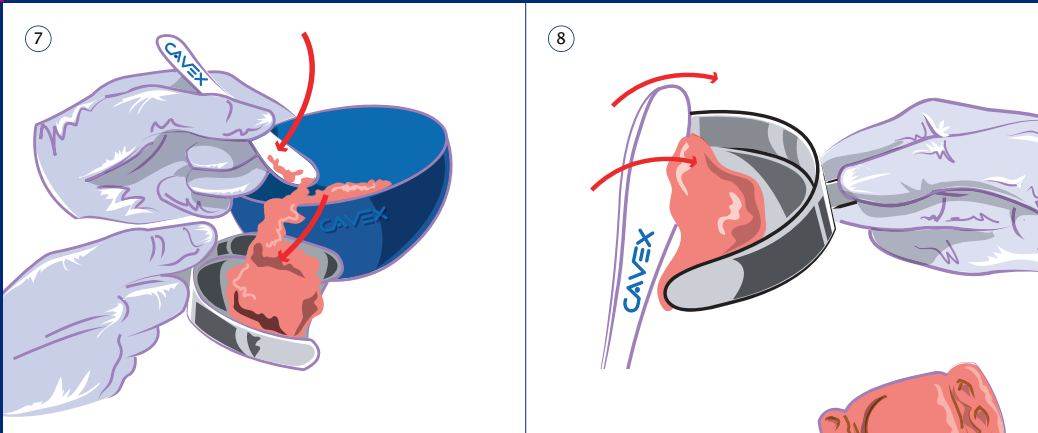


Step E

The dentist places the tray in the mouth, allows the paste to cure and removes the tray from the patient's mouth in one swift, supple movement.

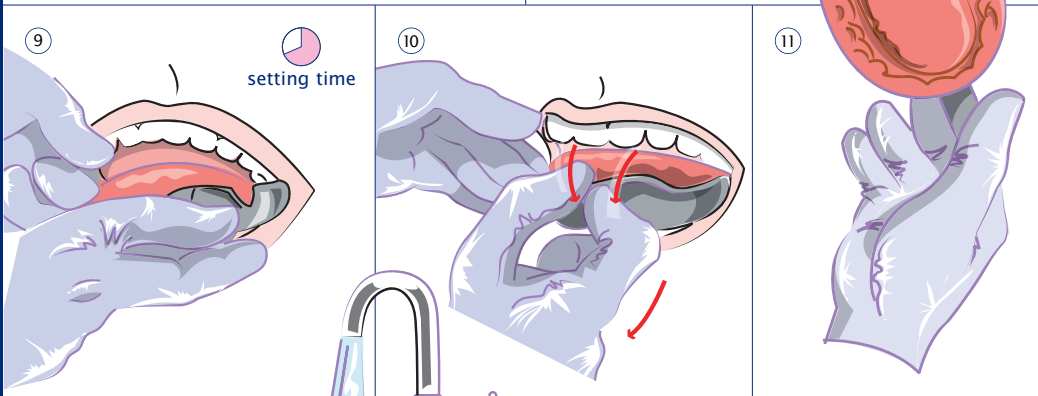

Step F

Rinse off the tray with the impression in cold water and tap off any superfluous water. Disinfect the impression, then rinse again and tap off on your hand.

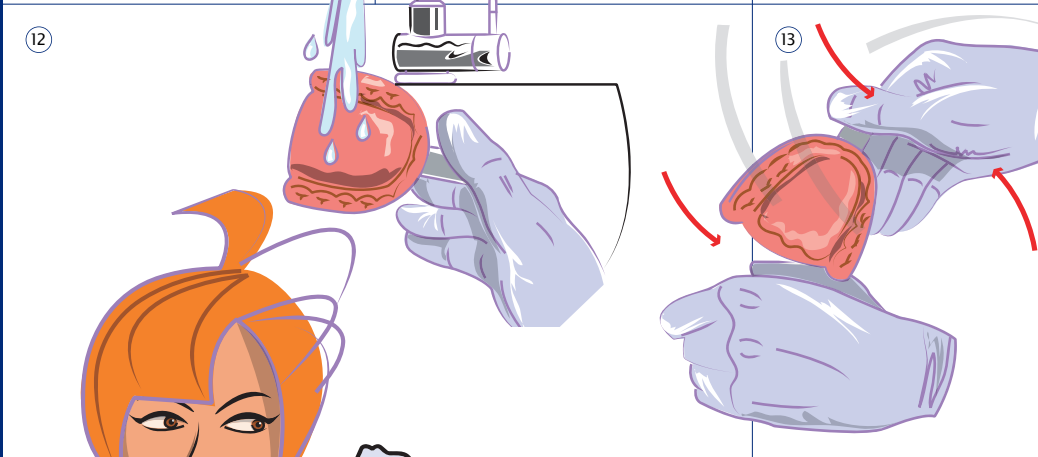

And then the final step, off to the laboratory

Place the still-damp tray in a plastic bag without adding any water, wet cotton wool or other moisturisers. Seal the bag hermetically, to retain the atmospheric humidity.

