

Cavex Alginate Mixer 3

Protocol

Cavex Holland BV
Fustweg 5
2031 CJ Haarlem
The Netherlands
T +31 23 530 77 00
dental@cavex.nl
www.cavex.nl



1. Put the right dosage of alginate and water in the mixing cup.



2. Stir shortly to make cleaning the mixing cup easier and preserve the mixer.



3. Make sure the mixing cup is closed properly by turning the lid clockwise.



4. Place the mixing cup in the mixer and close the lid.



5. Select program, press start and wait for the program to finish.



6. Remove the alginate mix from the mixing cup in a single movement and apply it to the tray.

Maintaining and cleaning the Cavex Alginate Mixer 3:

Prevent contamination of the mixer by cleaning it regularly. When dirt comes loose, it can get into the motor and damage the mixer.

1. Dissolve some Cavex GreenClean in a bowl of water.
2. Moisten a cloth (not too wet!) with the GreenClean solution and clean the mixer.
3. Use the stirrer stick with the moist cloth to clean hard-to-reach places.



Recommended dosage upper jaw

Recommended dosage lower jaw

Accessories

