

## Cavex ImpreSafe disinfection protocol

For the disinfection of alginate, polyether and silicone impressions from bactericide, fungicide and virucide.

Mixing ratio: 30 ml. disinfectant on 1 l. water  
 Disinfection time: 3 min.



① Squeeze the bottle to fill the integrated measuring cup to 30 ml. with disinfectant.



⑤ Submerge the impression in the disinfection fluid for three minutes.



② Fill the container with water up to the indicator (1 liter).  
 (see TIP1)



⑥ After three minutes take the impression out of the disinfectant fluid.



③ Mix the disinfectant with the water in the container.



⑦ Rinse the impression with running water.



④ Place the impression on the tray in the container.



⑧ Shake off the excess water, put the impression in an airtight sealed bag and mark the bag with 'Disinfected: Yes!'.

**TIP1:** To prevent foaming, fill the container with water before adding the disinfection fluid.

**TIP2:** Refresh the disinfection fluid weekly or when it looks contaminated.