

Improving impressions, reducing deformations

Cavex Alginate Adhesive




Features and benefits:

- Improves both physical and chemical adhesion between impression tray and alginate impression material.
- Less deformation to impression will occur when removing tray from mouth, thus improving overall quality of the impression.
- Exclusively developed for use with alginate impression material on both metal and resin trays.
- Every bottle cap is equipped with a small applicator brush.
- Applicator makes it easy to target adhesive.
- Applicator generates less hazes in comparison to sprays.
- Distinctive blue colour makes adhesive easy to work with.
- Over 50 applications per bottle.
- Residue of Cavex Alginate Adhesive is easily removed using isopropyl alcohol or ethanol (95%).



Improving impressions, reducing deformations

Cavex Alginate Adhesive adhesion in combination with Cavex CA 37.

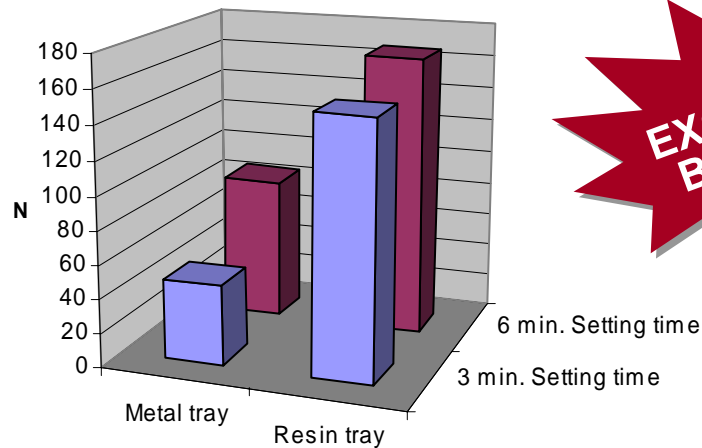

	Metal tray	Resin tray
3 min. Setting time	47	151
6 min. Setting time	85	167

More indepth information about this study is available on file.



Instructions for use:

1. Apply thin layer of Cavex Alginate Adhesive on the impression tray. Wait 3 to 5 minutes.
2. Apply alginate impression material.
3. Take impression like normal.
4. Remove alginate using Cavex GreenClean.
5. Clean tray with isopropyl alcohol or ethanol (95%).

