



*Impression materials are the starting point of many dental operations. High-quality impressions give the dentist confidence in a good end result. Cavex offers a well-trying and reliable range of alginates and auxiliary materials developed according to the requirements of the dental practice.*

*Your impression is our concern*

## **A stable full denture thanks to the gentle approach of the soft tissue.**

The fit of a dental prosthesis is the most decisive factor in its functionality; the base must form a snug fit with the soft tissues in the mouth. Cavex Outline is developed specially to achieve this. A non-elastic impression paste with required flow properties for an edentulous jaw, that ensures an accurate impression of the soft tissue.





## Accurate impressions, Eugenol-free.

- **ideal consistency for edentulous jaws** With the introduction of the new Cavex Outline, eugenol irritation of the soft tissue now becomes a thing of the past. Cavex Outline produces an accurate registration for the relining, rebasing and making of a full prosthesis, without the risk of irritation and odourless for both patient and dentist.

- **non-elastic**

- **intra-oral snap set**

- **fully cured, optimal final hardness**


- **not brittle**

- **not sticky, no film residue**

- **3 years shelf-life**

## Ideal consistency, optimal final hardness.

The paste flows easily under light pressure – thixotropic - so the soft tissues are not distorted. There is sufficient time to simulate the various mouth movements. Cavex Outline is also non-elastic, so the material does not bounce back when the tray is removed from the mouth. The result is a highly legible impression (blue), with all details clearly visible.

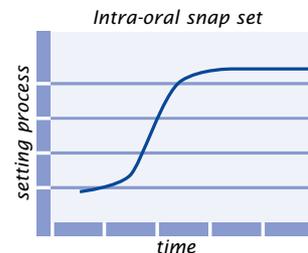

The moisture within the patient's mouth accelerates the curing process (intra-oral snap set). In approximately 3 minutes you have a firm, fully-cured impression that you can cut without crumbling.

## Not sticky, easy dosage and economical.

Cavex Outline does not leave any residue on the palate or the soft tissue in the mouth. Gypsum compatibility is excellent. The impression does not shrink and remains dimensionally stable for at least 7 days.

Both components – the base (blue) and the catalyst (white) – are economical in use and are easily dosed from the new laminated tubes. An adhesive is unnecessary. Both pastes have a shelf-life of 3 years.

Cavex Outline is available from your dental supplier. For further information on product composition and applications, please contact Cavex, +31 23 530 77 00 or [www.cavex.nl](http://www.cavex.nl)

