

Quadrant FiniSense:

STOP sensitivity!

- For pain caused by exposed dentine
- Prevention and relief of post-operative pain after crown and inlay preparation


Why Quadrant FiniSense?

- effective within 30 seconds
- works longer then 12 months
- leaves no film/residue on surface
- easy to use
- 150 applications per bottle



How does it work?

⇒ Immediate intradentinal closing stops fluid currencies in the tubuli

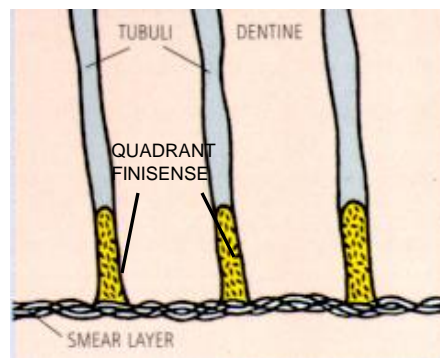


Result:


Prevention/relief of post operative pain

Quadrant FiniSense Desensitizing Primer

- For pain caused by exposed dentine
- Prevention and relief of post-operative pain after crown and inlay preparation

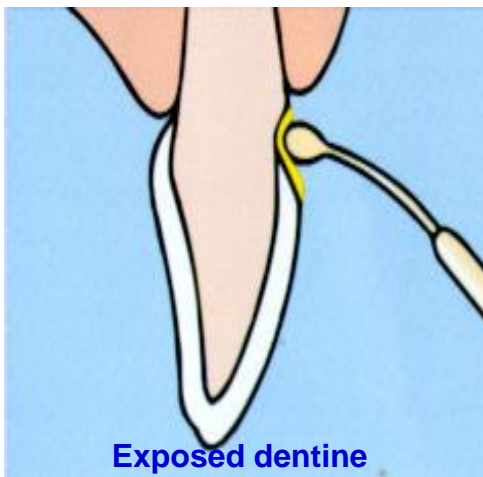

- Clean the dentine with for instance pounce
- Carefully dry the dentine surface
- Apply Quadrant FiniSense and let it act on surface for 30 sec.
- Blow dry gently
- Rinse with water thoroughly

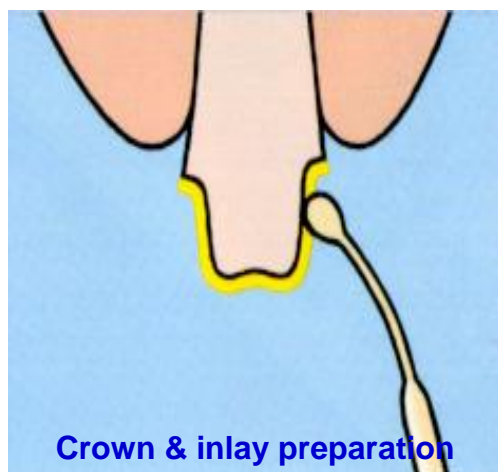

- Carefully dry the dentine surface
- Apply Quadrant FiniSense and let it act on surface for 30 sec.
- Blow dry gently
- Rinse with water thoroughly