

# Cavex Alginate Container

Several external factors may influence the quality of the alginate impression. It is therefore essential to standardise as many of these factors as possible. The Cavex Alginate container is an accessory that is part of the solution to standardise external factors.

## How to use

Keep the alginate powder in the container and stick the included sticker with the relevant alginate on the container. Scoop the alginate powder out of the container by using the included spoon. Scrape the spoon to the side of the container for the perfect amount of alginate powder.



### Single hand

The Alginate Container can be handled with one hand. Pull the lid and use the spoon, attached to the inside of the lid, to get a scoop of alginate powder.



### Air tight/Light tight

Deterioration due to light or moisture is avoided by making the container air- and light-tight.



### Dosing spoon

The dosing spoon can be snapped onto the inside of the lid of the container. In this way the spoon is never lost or covered under alginate powder.

## CAVEX YOUR IMPRESSION IS OUR CONCERN

