

P R E M I U M O R A L C A R E

-

CAVEX ORAL PRE RINSE

-

Cavex Oral Pre Rinse contains 1,5% hydrogen peroxide. The oxygenating action of the hydrogen peroxide works as a bleaching agent. It reduces the total amount of microorganisms and removes discolorations from the teeth.

-

Indications:

Cavex Oral Pre Rinse is a pre procedural rinse for intra-oral dental treatment (i.e. whitening).

-

Ingredients:

Aqua, erythritol, glycerol, hydrogen peroxide, poloxamer, PVM/MA copolymer, mint aroma, sodium saccharin.

-

Directions:

Cavex Oral Pre-Rinse is a ready-to-use rinse aqueous solution. Patients 18 years of age and older rinse 10 – 15 ml for at least 30 seconds, then thoroughly spit out.

Always follow directions from dentist or dental professional.

-

Safety warning:

For professional use only. Avoid contact with eyes. Rinse eyes immediately if product comes in contact with them. Contains hydrogen peroxide. Do not swallow.

-

Content:

500 ml aqueous solution.

-

Storage:

Unopened bottles: 1 year when refrigerated.

After opening: 1 month at room temperature.

YOUR
IMPRESSION
IS OUR
SPECIALTY



CAVEX