

CAVEX BITE&WHITE ABC MASTERKIT

WOULD YOU LIKE A BRIGHT WHITE SMILE?

Recommended by professionals

Cavex Bite&White is a product of Cavex Holland, which specialises in the manufacture of dental products and has more than a hundred years of experience in this field. The Cavex Bite&White Professional Dental Whitening System has been extensively tested, and is recommended by dentists and dental hygienists. Cavex Bite&White contains 16% carbamide peroxide (= 6% hydrogen peroxide)*, a substance with a proven reputation as a whitening agent. It is effective without damaging your teeth.

Fluoride

Cavex Bite&White whitening gel contains sodium fluoride to reinforce the enamel. It also contains potassium nitrate, which helps to prevent sensitivity. A mint flavour has been added to improve the taste, just in case a small amount of the material should come out of the tray. A qualified dentist should be consulted prior to performing a whitening treatment. This is to eliminate the presence of any oral or other health problems and to ensure the correct use of Cavex Bite&White. All Cavex Bite&White whitening materials are pH-neutral (pH = 6.2) which means that they are safe for your teeth.

More information

Your dentist can provide further information about Cavex Bite&White.

In addition to the Cavex Bite&White ABC Master kit, Cavex Bite&White refills, Cavex Bite&White ExSense and Cavex Bite&White StainLess are also available as separate products.



* Hydrogen peroxide \leq 6 % should not be used by persons under the age of 18; the product and treatment are only available through dental care professionals and for the period specified.



- Safe, effective and easy to use
- Your dentist will draw up your treatment plan, take impressions and supply the tray
- Home whitening with fast results
- Complete whitening treatment

CAVEX
Bite&White
Professional Dental Whitening System
ABC MASTERKIT

■ Cavex Holland BV
Fustweg 5
2031 CJ Haarlem
The Netherlands

+31 (0)23 530 77 00
dental@cavex.nl
www.cavex.nl

ZW11E.01

Home
whitening
under the
supervision of
your dentist

pH-neutral -
safe for teeth!



CAVEX
Bite&White
Professional Dental Whitening System
ABC MASTERKIT



Bright white teeth – safely and quickly – thanks to the Cavex Bite&White ABC System

Bright and white

It is so nice to see beautiful white teeth. It's enough to make you envious! Having white teeth is a great boost to anyone's confidence. And did you know that with Cavex Bite&White – a completely safe product for home whitening – it is quite easy to achieve these days? Cavex Bite&White is quick-acting and easy to use, and it will give you back your gleaming white smile!

Safe and effective tooth whitening

Most people begin life with perfect white teeth. However, these may discolour in the course of time due to the use of coffee, tea and other foodstuffs as well as some medicines, and hereditary dental issues may also play a role. And although we may be reluctant to admit it, getting older also has an effect on the colour of our teeth. The protective enamel inevitably gets thinner, increasingly revealing the relatively dark, underlying dentine and allowing discolouring substances to penetrate even faster.

Whitening the teeth is the obvious solution and Cavex Bite&White has been developed to do just that. This is a safe and effective whitening agent. Treatment does not take place in a clinic or in the dentist's chair; you can perform the treatment yourself, wherever and whenever it suits you. Your teeth can once again be just as white as those of that person you were always so envious of.*

* Results will be dependent, among other things, upon the nature and intensity of the discolouration, overall oral hygiene and the duration of the treatment.

Consult your dentist

Only dental practitioners are allowed to dispense Cavex Bite&White. It is advisable to consult your dentist before whitening your teeth; he or she can provide detailed information and useful tips about the process, as well as doing some essential preparatory treatment, such as polishing your teeth and removing tartar. Your dentist will also be able to establish the current colour and select a new colour in keeping with your wishes. The colour selected will provide the basis for the whitening schedule.



Prior to treatment.



After treatment: a gleaming smile.

Get started safely – with a well-fitting tray

A custom-fitted tray is made by the dentist using a professional imprint of your teeth. It will fit perfectly around all your teeth, ensuring that the whitening gel will barely touch your gums, if at all. Your dentist will specify the exact amount of gel to be used.



You will then take the transparent tray and whitening gel home to use.

Home whitening – as easy as ABC!

The Cavex Bite&White ABC system ensures the safe and effective whitening of your teeth – without sensitivity – in 3 steps. The system consists of Cavex Bite&White StainLess, Cavex Bite&White whitening gel and Cavex Bite&White ExSense.

Step A: Cavex Bite&White StainLess

Your teeth are thoroughly cleaned by the intensive action of this toothpaste. As well as removing superficial stains, it also raises the acidity in your mouth, which increases the effectiveness of the whitening gel. All in all, this toothpaste ensures a perfect start to your whitening treatment!



Step B: Cavex Bite&White whitening gel

The scientifically proven gel ensures a safe and speedy whitening result. The tray containing the gel is kept in the mouth for about 45 minutes.



Step C: Cavex Bite&White ExSense

Cavex Bite&White ExSense is a post-whitening conditioner for your teeth. It is used after the whitening treatment to re-mineralise the teeth and ensure improved hardness. It also prevents sensitivity – if any. The conditioning gel is applied in the tray, and is kept in the mouth for no longer than five minutes.

